

# GE Venue GO General Instructions

UABEM 8/14/2023


# Venue GO- Overview



1. AC mains power indicator & Battery status display
2. System identification and power status stripe indicator
3. System's Physical Power ON/OFF button

# Starting a new study


1.

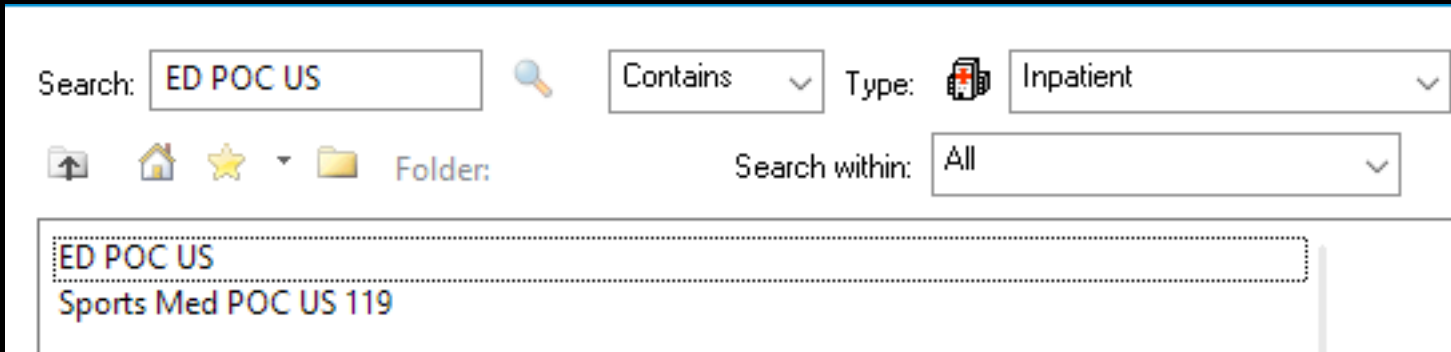

Search: ED POC US Contains Type: Inpatient Search within: All

ED POC US  
Sports Med POC US 119

2.

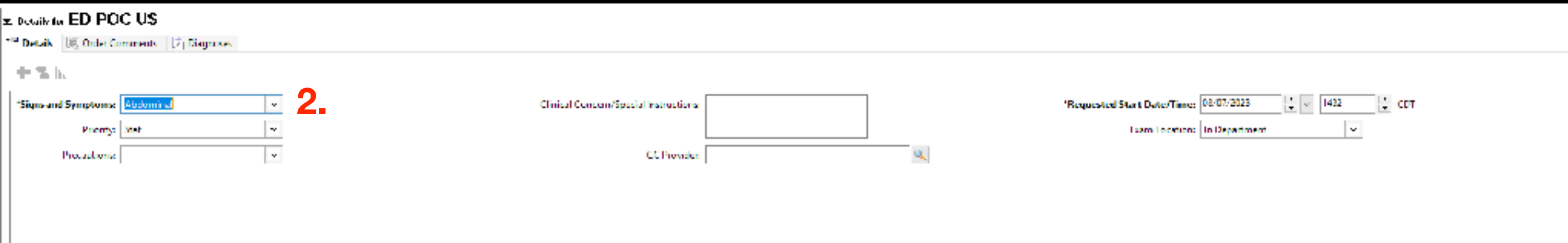

Details for ED POC US


Signs and Symptoms: Abdominal Priority: Visit Prevalence: Clinical Concern/Special Instructions: CC Provider: Requested Start Date/Time: 08/07/2023 1432 CDT Exam Location: In Department

1. Order “ED POC US” in EMR

2. Select single most important indication under “signs and symptoms” field

# Starting a new study


3.

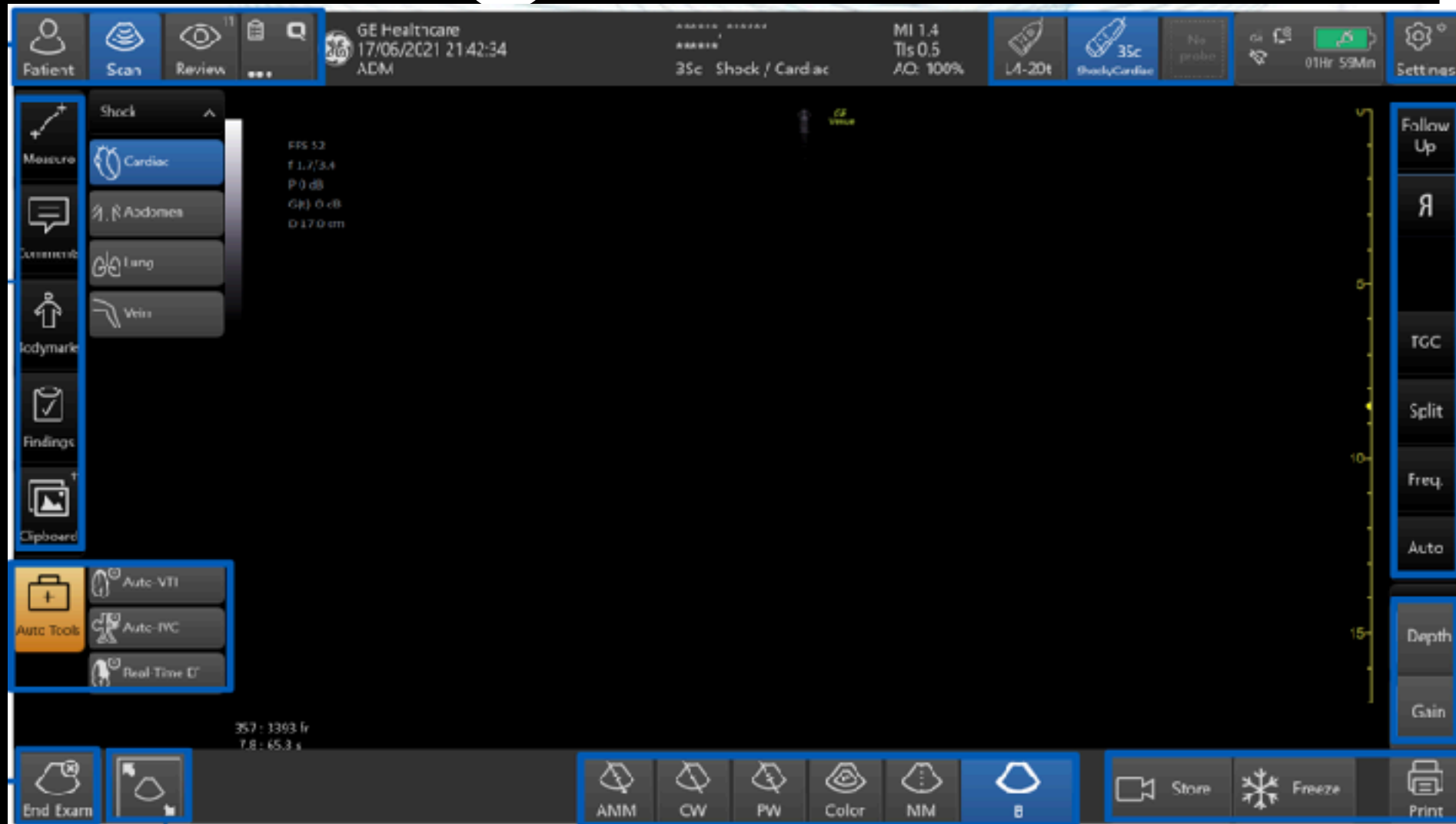

3. Select “Patient” in top left corner to enter new patient information\*

\*note: if a prior exam has not been ended, select “End Exam” prior to entering into a new study

# Starting a new study

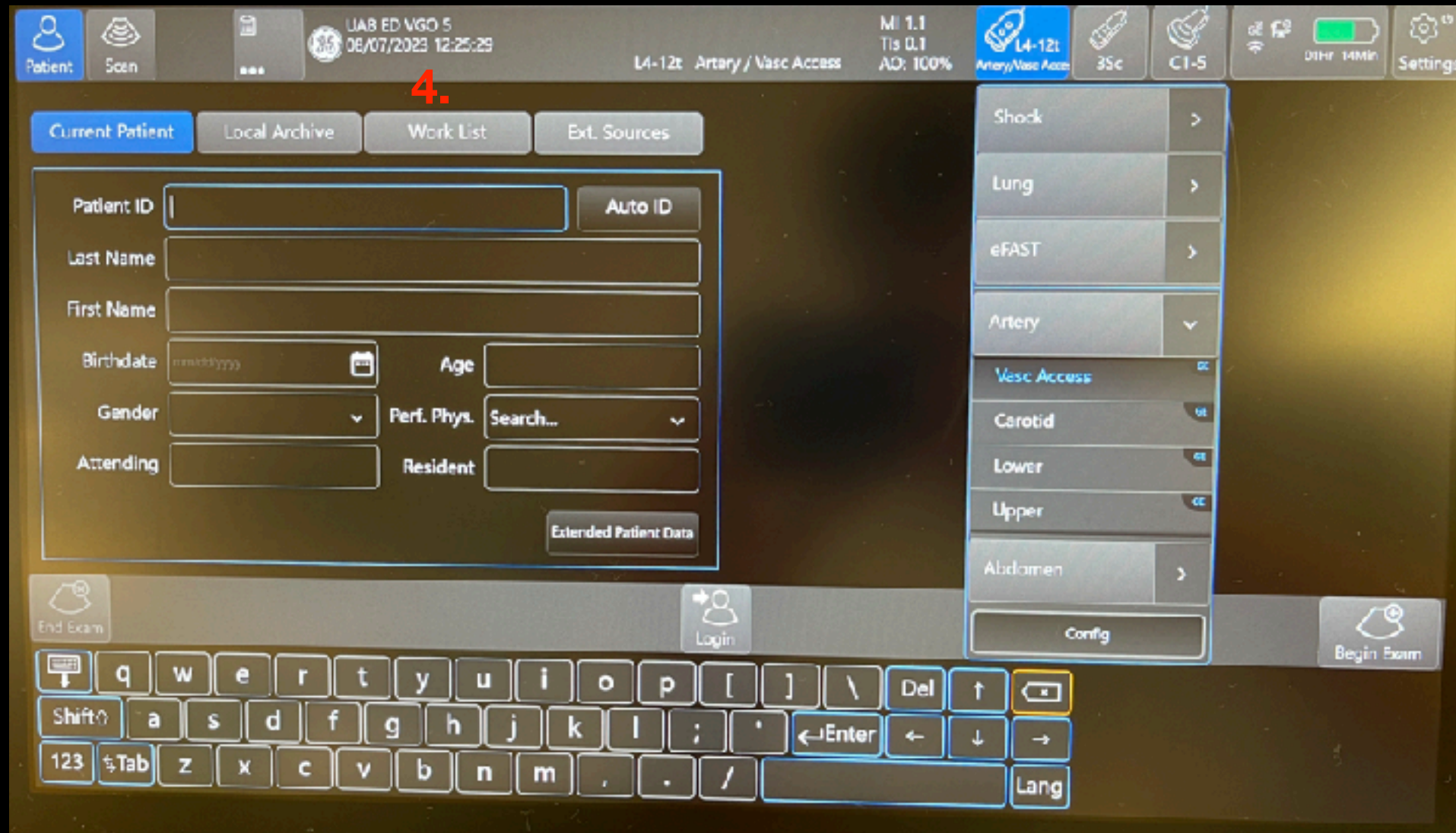

4. Select "Work List"



# Starting a new study

The screenshot shows a medical ultrasound software interface. At the top, there is a status bar with various icons and text: 'Patient', 'Scan', 'UAB ED VGO 5', '08/07/2023 12:25:57', 'ADM', 'L4-12: Artery / Vasc Access', 'MI 1.1', 'TIs 0.1', 'AO: 100%', 'L4-12t', '35c', 'C1-5', 'UE 1.2', '07Hr 14Min', and 'Settings'. Below the status bar, there are four buttons: 'Current Patient', 'Local Archive', 'Work List' (highlighted in blue), and 'Ext. Sources'. Underneath these buttons, there are filter fields: 'Filter by Patient ID', 'Filter by Last Name', and 'Filter by First Name'. To the right of these filters are 'Clear search', 'From 08/07/2023', 'To 08/07/2023', 'Today's Exams', and 'Refresh' buttons. The main area contains a table with the following columns: 'Last Name', 'First Name', 'Patient ID', 'DOB', 'Exam Type', 'Accession No.', 'Sched. Date', and 'Sched. Time'. The table lists four entries, all with 'Exam Type' 'ED POC US' and 'Sched. Date' '08/07/2023'. At the bottom of the interface, there are buttons for 'End Exam', 'Logged in as ADM', 'Logout', and 'Begin Exam'.

Last Name	First Name	Patient ID	DOB	Exam Type	Accession No.	Sched. Date	Sched. Time
				ED POC US	US230050872	08/07/2023	11:33
				ED POC US	US230050863	08/07/2023	11:16
				ED POC US	US230050836	08/07/2023	10:25
				ED POC US	US230050828	08/07/2023	10:09

5.


5. Find the correct patient\*

6. Select "Begin Exam"

6.

\*The studies are displayed based on the most recent order. If you do not see any studies, be sure that the date is correct. Close to midnight you may need to press "Today's Exams" to find the correct patient.

# Starting a new study

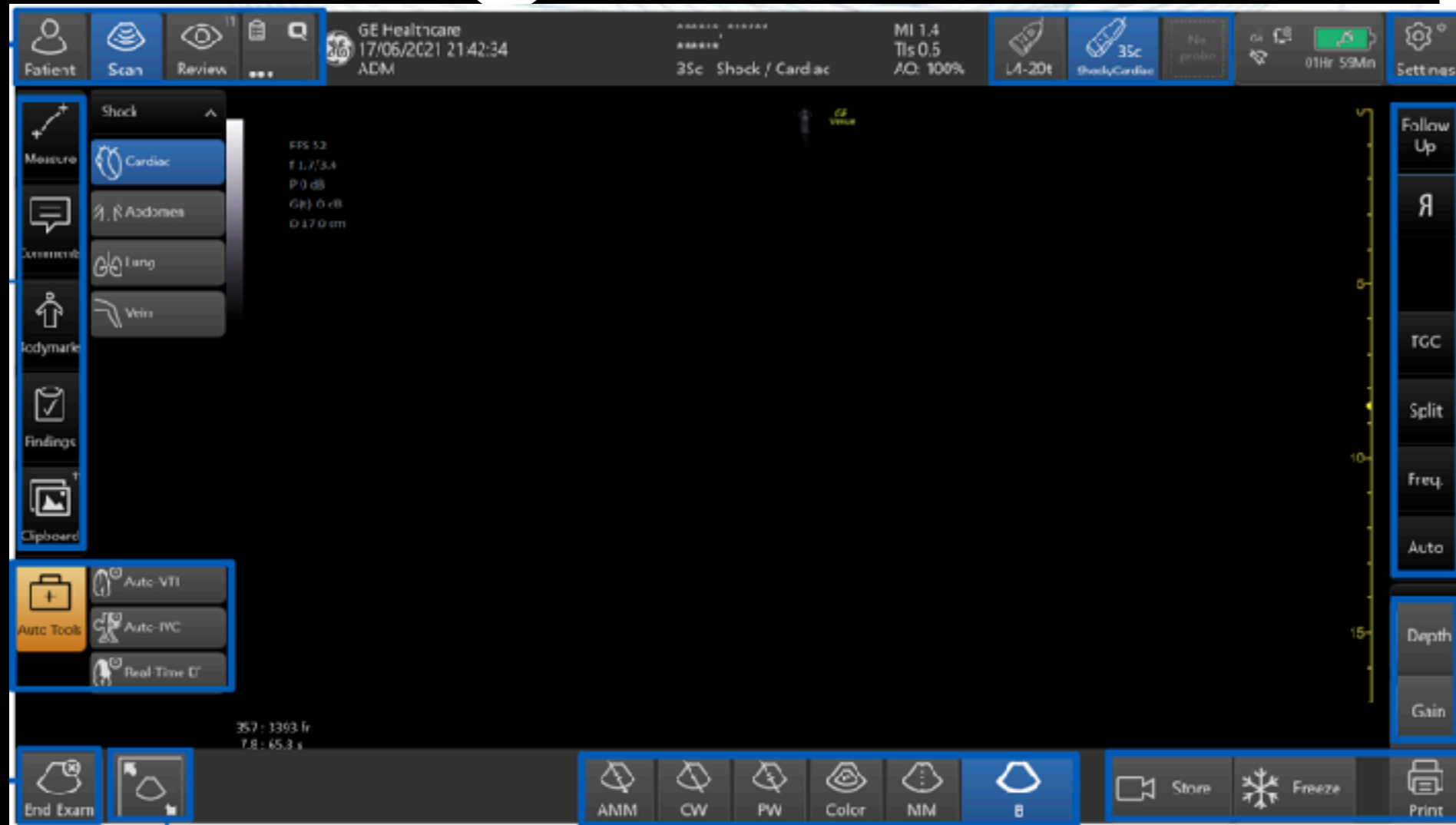

7. Scan away!

8. Select “End Exam” to complete your study

# FAQs




# “Help, the ultrasound is asking me to log in!”



The image shows a GE Login screen. At the top center is the GE logo, a circle containing the letters 'GE'. Below the logo is the word 'Login'. Underneath, there are two input fields: 'Operator' and 'Password'. The 'Operator' field is a dropdown menu with 'ADM' selected and a small downward arrow on the right. The 'Password' field is a simple text box that is currently empty. At the bottom of the screen, there are four buttons: 'Keyboard' (with a keyboard icon), 'Cancel', 'Emergency', and 'Login'.

- If you encounter this screen, just select “Login”. The password field is left blank.

# “I forgot to order the study”

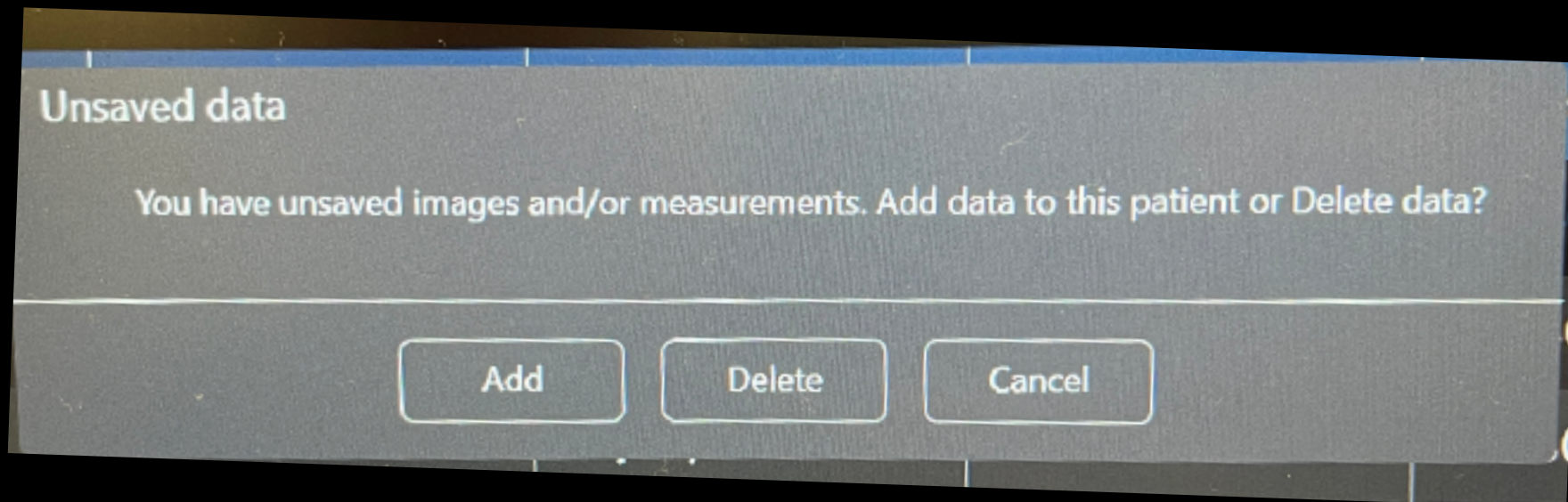

- If a study is done emergently and there was no time to put the order in before the patient was registered, you are able to add the information after the fact.
- Step 1: Do not end the study
- Step 2: Enter the order in the EMR as per usual.
- Step 3: Select Patient followed by Worklist and choose the correct patient (steps 3-6 above)
- Step 4: When prompted, choose the “Add” option
- End Study as usual